## Warranty and Disclaimers

Garantie et avis de non-responsabilité | Garantía y descargos de responsabilidad

# Limited Warranty and Disclaimer Information

Limited Warranty | Disclaimer and Venue

## LIMITED WARRANTY

Owlet warrants that the Owlet Smart Sock™ 2 will be free from defects in materials and workmanship, under normal use and service, for a period of one (1) year from the date of purchase as established by the purchase receipt. This Limited Warranty is only available to the original purchaser of this Owlet Smart Sock 2.

At its option, Owlet may repair or replace the defective Owlet Smart Sock 2 or defective component part(s) in accordance with the terms of this Limited Warranty Period. The replacement unit will be warranted only for the unexpired period of the original monitor's Applicable Warranty Period. This Limited Warranty does not apply if the Owlet Baby Monitor has been damaged as a result of misuse, modifications, or improper maintenance. This Limited Warranty is the only warranty for the Owlet Smart Sock 2. No one is authorized to make any other warranty on behalf of this monitor. Some states or countries do not allow limitations on how long an implied warranty lasts or for the exclusion of incidental or consequential damages, so these limitations or exclusions may not apply to you. This Limited Warranty gives you specific legal rights, and you may also have other rights depending on the laws of your state or country.

## DISCLAIMER AND VENUE

**The Owlet Smart Sock and Owlet App are intended to act as an aid in caring for your baby, and are not intended to replace, be used without, or in lieu of, monitoring by a responsible adult.**

By using the Owlet Smart Sock 2, the user agrees and affirms that Owlet Baby Care Inc. including it's agents, employees, investors, and members shall not be held liable for any loss, damage, or injury arising in connection with the use or operation of the Owlet Baby Monitor, regardless of cause.

The Owlet Smart Sock 2 is meant for healthy babies; therefore, the notification thresholds on the Owlet Smart Sock 2 are not as stringent as a medical monitor.

Remember:

- Do not use Owlet Smart Sock 2 as a diagnostic tool
- Do not use Owlet Smart Sock 2 if your doctor recommends the use of a hospital pulse oximeter
- Do not use Owlet Smart Sock 2 if your doctor recommends the use of a hospital apnea monitor
- Do not use Owlet Smart Sock 2 as an excuse for unsafe sleep habits

By using the Owlet Smart Sock 2, the user agrees and affirms that any dispute arising out of its use or operation shall be brought in and heard by the Third Judicial District Court of Salt Lake County, State of Utah, USA.

### READ THIS MANUAL

Inside you will find many helpful hints on how to use and maintain your Owlet Smart Sock 2 properly. The troubleshooting section provides answers to some problems. If you review the troubleshooting tips first, you may not need to contact us for assistance. Complete setup and use information is available online at [www.owletcare.com/setup](http://www.owletcare.com/setup)

### READ THE SAFETY INFORMATION

Your safety and the safety of others are very important to us. There are many important safety messages in this manual and in the Owlet Baby Care App. Always read and obey all safety messages.

**This symbol is used in this manual as an indication of Important Safety Information!**

All safety messages will follow the safety alert symbol and either the word "DANGER", "WARNING", "CAUTION" OR "NOTICE". These words mean:

**DANGER** = An imminently hazardous situation that will result in death or serious injury.

**WARNING** = A potentially hazardous situation that could result in death or serious injury and/or damage to property.

**CAUTION** = A potentially hazardous situation that may result in minor or moderate injury.

**NOTICE** = Hazards that could result in things other than personal injury, like property damage.

## WARNINGS AND IMPORTANT SAFETY INFORMATION

**WARNING: The Smart Sock 2 is not a medical device.** The Owlet Smart Sock 2 notifies you if your baby's heart rate or blood oxygen levels are outside of the provided range. The Smart Sock 2 is designed

for newborns to eighteen (18) months not for after eighteen (18) months. The Owlet Smart Sock 2 is not a medical device. It does not

- Diagnose, cure, lessen, monitor, treat, nor prevent disease or injury;
- Affect the function or structure of the body;
- Achieve primary intended purposes through chemical action;
- Does not prevent Sudden Infant Death Syndrome (SIDS).

This manual serves two primary functions: To provide the user with the basic directions and recommendations for the proper installation and use of the Owlet Smart Sock 2; and, to explain features, operation, safety precautions, maintenance, and troubleshooting of the Owlet Smart Sock 2. It is essential that all persons who are expected to operate or adjust this Owlet Smart Sock 2 read the instructions carefully, that they understand how to properly operate the Owlet Smart Sock 2, and that follow these instructions.

If you do not understand these instructions, please contact us right away at [contact@owletcare.com](mailto:contact@owletcare.com) or 1 844 334-5330 from 7AM - 9PM (MST) Mon. - Sat.

### FOR YOUR SAFETY AND THE SAFETY OF THE BABY

• ALWAYS KEEP THE BASE STATION AT LEAST THREE FEET AWAY FROM YOUR BABY'S CRIB BECAUSE THE CORD CAN POSE A STRANGULATION HAZARD.

- THE OWLET BABY SMART SOCK 2 IS AN AID. IT IS NOT A SUBSTITUTE FOR ADULT SUPERVISION.
- FOR PREMATURE BABIES, BABIES ON SUPPLEMENTAL OXYGEN, OR BABIES WITH HEALTH CONDITIONS, PLEASE CONSULT YOUR HEALTH PROFESSIONAL TO DETERMINE WHETHER OR NOT THE OWLET SMART SOCK 2 IS RIGHT FOR YOU.
- RESPOND IMMEDIATELY WHENEVER A NOTIFICATION SOUNDS.

## FCC COMPLIANCE STATEMENT

Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment. Please ONLY use the supplied power adapter. Failure to do so may void our limited warranty. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the manufacturer's instructions, may cause interference harmful to radio communications.

This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interferences, and (2) this device must accept any interferences, including interferences that may cause undesired operation of the device.

# Troubleshooting Guide

Initial Setup | Notifications | Connectivity

## Initial Setup

| Problem                                                                                                            | Causes                                                                                                                                                                           | Solutions                                                                                                                                                                                                                                                                                                                                                                                                 |
|--------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <ul style="list-style-type: none"><li>• Cannot see the Smart Sock 2 Wi-Fi during setup</li></ul>                   | <ul style="list-style-type: none"><li>• The Base Station is not turned on</li><li>• Already connected to Smart Sock 2 Wi-Fi</li><li>• The Base Station is out of range</li></ul> | <ul style="list-style-type: none"><li>• Plug in the Base Station</li><li>• Check if the Wi-Fi light on the bottom of the Base Station is on. If so, you can skip this step and continue setting up your account in the app</li><li>• Move the Base Station closer to your Wi-Fi router</li></ul>                                                                                                          |
| <ul style="list-style-type: none"><li>• Cannot see your home Wi-Fi in the Owlet App</li></ul>                      | <ul style="list-style-type: none"><li>• Your home router is 5.0 GHz (Owlet only supports 2.4 GHz)</li><li>• You may have a hidden router</li></ul>                               | <ul style="list-style-type: none"><li>• Make sure your iOS device is connected to a 2.4 GHz network (most 5.0 GHz routers also support 2.4 GHz)</li><li>• Visit <a href="http://owletcare.com/support">owletcare.com/support</a> for instructions on hidden routers</li></ul>                                                                                                                             |
| <ul style="list-style-type: none"><li>• The Base Station will not connect to your home Wi-Fi</li></ul>             | <ul style="list-style-type: none"><li>• Password is incorrect</li><li>• The Base Station is out of range of your Wi-Fi router</li><li>• Your home Wi-Fi is not working</li></ul> | <ul style="list-style-type: none"><li>• Double check your internet password</li><li>• Move the Base Station closer to your internet router for setup, then move the Base Station back to the nursery after set up</li><li>• Try restarting the phone and reconnecting to your Wi-Fi</li><li>• Reset your Wi-Fi router</li></ul>                                                                           |
| <ul style="list-style-type: none"><li>• The Base Station remains white after the Smart Sock is unplugged</li></ul> | <ul style="list-style-type: none"><li>• Software update is in progress</li></ul>                                                                                                 | <ul style="list-style-type: none"><li>• Plug the Smart Sock back into the Base Station until the update is complete</li></ul>                                                                                                                                                                                                                                                                             |
| <ul style="list-style-type: none"><li>• The Base Station will not pair to your Owlet account</li></ul>             | <ul style="list-style-type: none"><li>• The Base Station and your iOS device are not on the same Wi-Fi network</li><li>• Holding the small Register Button</li></ul>             | <ul style="list-style-type: none"><li>• Verify that your iOS device is connected to same Wi-Fi network as the Base Station</li><li>• Press and release (do not hold) the small Register Button on the bottom of the Base Station and wait a few seconds before trying to pair your Owlet account again</li><li>• Connect your iOS device to the same 2.4 GHz Wi-Fi network as your Base Station</li></ul> |

\*Need Technical Support? Email us at [contact@owletcare.com](mailto:contact@owletcare.com) or call us at +1 (844) 334-5330 from 7AM - 9PM (MST) Mon. - Sat.  
For all general questions chat with us at [owletcare.com/chat](https://owletcare.com/chat)

## Notifications

| Problem                                                                                                | Causes                                                                                                                                                                                                                                                                             | Solutions                                                                                                                                                                                                                                                                                                                                                                                             |
|--------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <ul style="list-style-type: none"> <li>The Base Station is flashing blue and making noise</li> </ul>   | <ul style="list-style-type: none"> <li>The Base Station is out of range of the Smart Sock</li> <li>The caretaker's body is blocking the Smart Sock to Base Station connection. This is common during feedings, diaper changes, snuggles, etc.</li> <li>Low/dead battery</li> </ul> | <ul style="list-style-type: none"> <li>Move the Base Station closer to the Smart Sock</li> <li>Reposition your baby or consider turning off monitoring during feedings, diaper changes, snuggles, etc.</li> <li>Recharge the Smart Sock</li> </ul>                                                                                                                                                    |
| <ul style="list-style-type: none"> <li>The Base Station is flashing red and making noise</li> </ul>    | <ul style="list-style-type: none"> <li>Low/high heart rate</li> <li>Low oxygen levels</li> <li>The Smart Sock does not fit properly</li> </ul>                                                                                                                                     | <ul style="list-style-type: none"> <li>Make sure the notch is on the outside of your baby's foot just 1-2mm behind the pinky toe.</li> <li>Make sure the Smart Sock is snug (if not, it is difficult to get a reading. False notifications could occur)</li> <li>Make sure the sensors are centered in the windows of the Fabric Sock</li> <li>Consider using a different Fabric Sock size</li> </ul> |
| <ul style="list-style-type: none"> <li>The Base Station is flashing yellow and making noise</li> </ul> | <ul style="list-style-type: none"> <li>Difficulty getting a reading</li> <li>The Smart Sock was placed incorrectly</li> <li>The lighting in the room is too bright</li> </ul>                                                                                                      | <ul style="list-style-type: none"> <li>Make sure the notch is on the outside of your baby's foot just 1-2mm behind the pinky toe.</li> <li>Make sure the Smart Sock is snug (if not, it is difficult to get a reading, and more false alerts will occur)</li> <li>Make sure the sensors are centered in the windows of the Fabric Sock</li> <li>Dim the lighting in your baby's room</li> </ul>       |

## Connectivity

| Problem                                                                                              | Causes                                                                                                                                                                                           | Solutions                                                                                                                                                                                                                                                                                                          |
|------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <ul style="list-style-type: none"> <li>The Base Station is disconnected from the internet</li> </ul> | <ul style="list-style-type: none"> <li>The Base Station is out of range of your Wi-Fi router</li> <li>Your home Wi-Fi is not working</li> </ul>                                                  | <ul style="list-style-type: none"> <li>Move the Base Station and the Wi-Fi router closer together (neither Wi-Fi range extenders nor second routers will help)</li> <li>Reset your Wi-Fi router</li> </ul>                                                                                                         |
| <ul style="list-style-type: none"> <li>Unable to charge the Smart Sock</li> </ul>                    | <ul style="list-style-type: none"> <li>The Smart Sock is not plugged in all the way</li> <li>The Smart Sock requires a reset</li> </ul>                                                          | <ul style="list-style-type: none"> <li>Plug the Smart Sock in to charge until you hear a click and it is blinking green</li> <li>Turn the Sock Electronics off, then back on.</li> </ul>                                                                                                                           |
| <ul style="list-style-type: none"> <li>Seeing dashes instead of heart rate and oxygen</li> </ul>     | <ul style="list-style-type: none"> <li>The Smart Sock is trying to get a reliable reading</li> <li>Too much movement</li> <li>Poor Smart Sock placement</li> <li>Poor Fabric Sock fit</li> </ul> | <ul style="list-style-type: none"> <li>Make sure the sensors are centered in the windows of the Fabric Sock</li> <li>Do not use rockers or vibrating pads, as they may interfere with the proper functioning of the device.</li> <li>Check the Smart Sock placement</li> <li>Check the Fabric Sock size</li> </ul> |

# Tips for a Safe Sleep Environment

Some friendly tips to keep your baby healthy and safe.

- Always place your baby alone and on their back to sleep. Placing your baby to sleep on their side or on their stomach is not safe.
- Use a crib that meets current safety standards. Use a firm mattress that fits snugly in the crib and is covered with only a tight-fitting crib sheet.
- Do not put anything soft, loose, or fluffy in your baby's sleep space. This includes pillows, blankets, comforters, bumper pads, stuffed animals, toys, or any other soft items.
- Use a wearable blanket, or similar type sleeper, instead of traditional blankets to keep your baby safe and warm.
- Room-share instead of co-sleeping. Keep your baby's sleep space separate from, but close to, your bed. This will keep them safe, and make feeding easier. The Smart Sock does not function properly with co-sleeping. Your body will interfere with the Smart Sock's wireless signal.
- Avoid falling asleep with your baby in your bed, couch, or armchair.
- Never allow your baby to sleep on any soft surfaces such as adult or child-sized beds, sofas, chairs, waterbeds, pillows, cushions, comforters, sheepskins, or anything similar.
- Do not use pillows, wedges, or positioners to prop your baby up, keep them on their back, or prevent them from rolling.
- Make sure your baby doesn't get too warm. Use light sleepwear on your baby and keep the room temperature comfortable for a lightly-clothed adult.
- Consider offering a pacifier to your baby at naptime and bedtime. For breastfed babies, delay introducing a pacifier until 3-4 weeks of age, when nursing is well-established.
- Educate everyone who cares for your baby about these rules for safe sleep!

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