
Smart Sock Quick Reference

The Owlet Smart Sock™ 2 uses two sensors to track your baby's readings.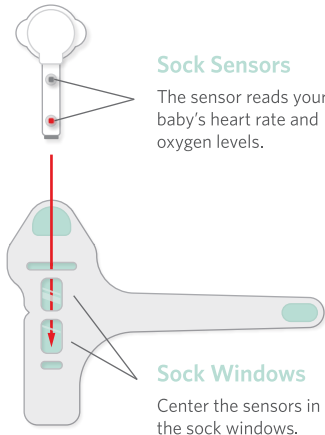

Interchangeable Sock

The Owlet sock can go on either the **right** or **left** foot. For proper placement on either foot, place the **notch** and **sock sensor** on the outside of the foot behind the pinky toe.


Right Foot

Dome on top of foot

Left Foot

Dome on bottom of foot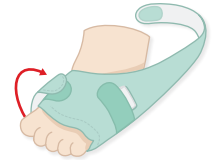

Sock Fit

Don't overtighten. If the strap goes past the velcro, try going down a size.

Base Station Quick Reference

See your baby's status with a glance at the Base Station

Sock Light

Green light indicates sock is connected to Base Station.

Registration

Push once when prompted during setup in the App.

WiFi Light

Green light indicate Base Station is connected to wifi.

On / Off


Press and release to turn off Base Station. The Base Station will beep, and its light will turn off.

Status Light

Light ring lets you know the status of your baby. More information is available in the App.

Dim Lights

Press and hold to switch between bright daytime mode to dim nighttime mode.

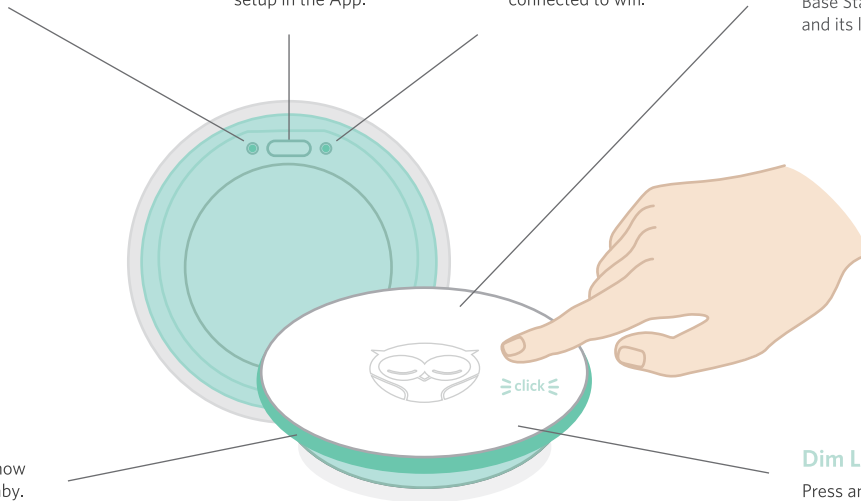
Base Station Color Reference


The Base Station has 5 different color status and notifications. To hear examples, go to the App.

Status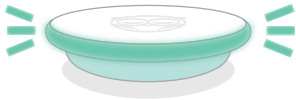


Blinking green light

Getting first heart rate and oxygen readings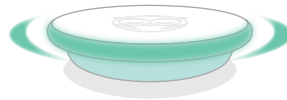

Slow pulsing green light


Heart rate and oxygen levels are in range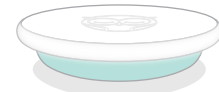

White static light


The Smart Sock is plugged in and charging

Notifications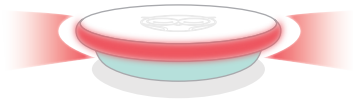


Red flashing light 🔊

Heart rate or oxygen levels are outside preset zone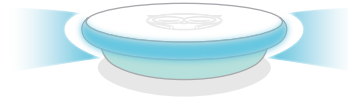

Blue flashing light 🔊

The Smart Sock is disconnected from the Base Station

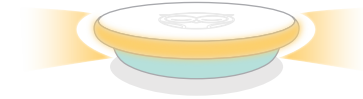Yellow flashing light 🔊

The Smart Sock fell off or is poorly placed, or shifted from proper placement

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